

# User Manual



## GTM-7101 Friction Strap



## Table of Contents

Introduction

Specification

Safety Instructions

Friction Strap Installation and Removal

General Service and Care

Warranty

---

## Introduction

The GTM 7101 Friction Strap is an innovative concept that allows the GTM-7100 handrim user to increase the grip friction of the handrim.

This uniquely increases the range of conditions under which the hand rim can be operated effectively.

The compact straps are easily installed or removed and therefore only used when required. They are easy to store and take along.

Should the strap be worn or damaged, it can be replaced independently of the handrim.

## Specification

Width:	30 mm
To fit handrim:	GTM 7100 Low Flyer
Mass per Strap:	55g $\pm$ 5 g
Material:	Reinforced polymer sheet



## Safety Instructions

### WARNING

Take careful note of the points below. If you do not, this may result in **serious injury and/or death**.

Only use the strap if it fits tightly on the handrim and is not slack or loose.

Any cuts, cracks or delamination may damage your hand or cause the strap to come off the handrim. Replace the strap immediately.

Do not stretch the strap more than required to put it on or take it off the handrim, this could stretch the strap causing it to fit loosely.

Ensure that the handrim is clean before installing.

At high speed, or when descending a hill, be aware that the strap will heat your hand faster. Adjust your speed and approach accordingly.

The black surface can get hot when exposed to direct sunlight. Check and allow to cool down before using.

Do not exceed 170°F (77°C) exposure to the strap. High temperatures can occur inside an automobile or exposure to an open flame or other heat sources, which may damage the strap.

Do not use solvents to clean, only wash with warm soapy water.

## Friction Strap Installation and Removal

1. Ensure that the handrim surface is clean and free of debris. Anything caught between the strap and the handrim could cause damage to both.
2. Do not stretch the strap more than required to slip it onto the handrim. This may cause slackness in the strap

3. Position the strap on the bottom of the handrim and then work it up both sides to the top equally. As mentioned in point 2, stretch the strap to a minimum.

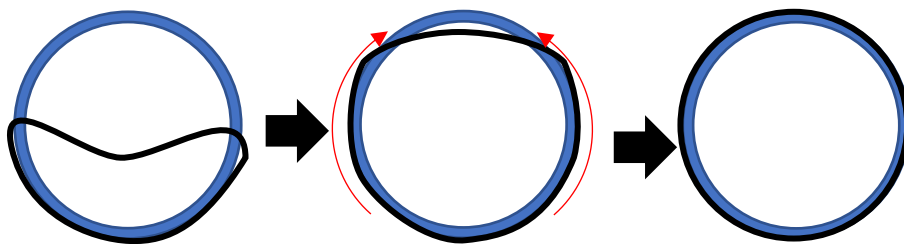


Figure 1. Installing the strap

4. Once on, position the strap that it aligns with the inner edge of the handrim.

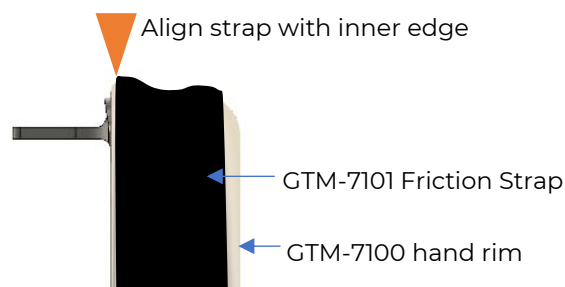


Figure 2. Alignment of strap

5. When removing the strap, do not use sharp items such as screw drivers. Either slide the strap off to the outside or insert a flat lever that will not damage the strap or handrim and then slide the strap off the handrim.



## General Servicing and Care

### STORING

Store strap in a cool dry place.

**Do not** position it near any heat or open flame sources.

### CLEANING

Clean the strap with a soft, moist cloth and a solution of dish soap and water.

**Do not** use industrial solvents or harsh chemicals that can damage the material.

### MAINTENANCE

The following guidelines are not considered to be a complete inspection. The guidelines will help maintain the performance and safety of the strap.

<b>Before use:</b>	<ul style="list-style-type: none"><li>• Check that the straps are not loose fitting on the rim.</li><li>• Check that the straps are not damaged.</li></ul>
<b>Weekly:</b>	<ul style="list-style-type: none"><li>• Clean the strap by wiping with a dish cloth and warm mild soapy water. Dry thoroughly before installation or storage.</li></ul>

Table 1: Maintenance



## Warranty

### WARRANTY

Globetrotter Motion warrants, to the original owner, that this GTM-7101 Friction Strap shall be free of defective materials and workmanship for a period of 6 months from the date of the original purchase (proof of purchase required) provided the GTM-7101 Friction Strap is operated under normal conditions and use. During this warranty period, Globetrotter Motion shall repair or replace, at its sole option, all parts that are found by Globetrotter Motion to be defective and subject to this Limited Warranty with this exception: Globetrotter Motion will offer a replacement for a GTM-7101 Friction Strap that is no longer in production / obsolete that otherwise fall within the time constraints of this warranty. This Limited Warranty is made only to the original owner of this new Globetrotter Motion GTM-7101 Friction Strap, and it shall remain in force only as long as the original owner retains ownership of the Globetrotter Motion GTM-7101 Friction Strap. This Limited Warranty is not transferable. This Limited Warranty does not apply to normal wear or tear, nor to defects, malfunctions or failures that result from the abuse, neglect, shocks or crashes, improper maintenance, alteration, modification, accident, or misuse, (including without limitation, basketball, quad rugby, tennis or any similar sporting activities), nor damage occurring during shipment, improper assembly or assembly with non-compatible products.

This Limited Warranty is the only express or Limited Warranty applicable to the Globetrotter Motion GTM-7101 Friction Strap. Any implied warranties, including warrants of merchantability and fitness for a particular purpose, shall be limited in scope and duration in accordance with this Limited Warranty. Globetrotter Motion shall not be responsible for any direct, incidental, consequential or exemplary damages suffered by any party. The foregoing statements of warranty are exclusive and in lieu of all other remedies. This Limited Warranty gives you specific legal rights; you may also have other legal rights which vary from state to state/country to country. Some states/countries do not allow limitations on the duration of implied warranties, or the limitations or exclusion of incidental or consequential damages; therefore, the limitations and exclusions set forth in this Limited Warranty may not apply to you. This Limited Warranty set forth herein may not be extended, enlarged or otherwise modified by any Globetrotter Motion dealer, distributor, agent, or employee, and Globetrotter Motion does not assume any liability or make any Warranty except as stated in this Limited Warranty. This warranty does not cover products whose serial number or identification has been erased, damaged or modified.

### FILING A WARRANTY CLAIM

File a warranty claim using our Warranty Claim Form under Support at <http://www.Globetrotter-motion.com>. We'll do everything we can to keep you mobile.